

Xiralite GmbH	
Quick Reference Guide Xiralite® X4	DOK-753-02-0ENUSA

Initial Investigation	Repeat Investigation	Image Reading
Turn on device and PC and start the program	Turn on device and PC and start the program	Turn on PC and start the program
“Case Selection”	“Case Selection”	“Case Selection”
	Select an existing patient and load the data	Select an existing patient and load the data
	Patient Data & Device	
“New Patient”	Parameters ⇒ “New to current patient”	
Enter patient data and operator name	Check operator information and change if needed	
Check device parameters	Check device parameters	
“Go to image acquisition”	“Go to image acquisition”	
Optional “Start Camera Preview” / “Stop Camera Preview”	Optional “Start Camera Preview” / “Stop Camera Preview”	
“Start Image Acquisition”	“Start Image Acquisition”	
Injection of ICG 10 s after starting the image acquisition	Injection of ICG 10 s after starting the image acquisition	
Set background signal; accept with "OK"	Set background signal; accept with "OK"	
Evaluate images	Evaluate images	
Open result page	Open result page	Open result page
Highlight the individual result using the left mouse button	Highlight the individual result using the left mouse button	Highlight saved result with the left mouse button
“Save Result”	“Save Result”	“Delete Result”
“Print Results”	“Print Results”	“Save Results”